**ZAMST ARM SLEEVE
INSTRUCTIONS FOR USE**

Please read through the following instructions completely before using the product.

Intended use: Arm protection

* This product can be used on the left or right arm.

* This product is not for medical treatment.

* This product is for arm protection; however, it will not prevent all arm injuries.

Do not use this product in the following cases:

- If you have a circulatory deficiency such as hypertension or heart disease. Your symptoms could worsen.
- If you have an arm ailment such as those caused by nervous disorders or diabetes. Your symptoms could worsen.
- If you have an allergic tendency and/or sensitive skin. (The product may cause contact dermatitis.)
- If you have abnormal conditions such as cuts, fractures, numbness, swelling or rash in the area where product is used.

Safety precautions:

- Do not use this product on any body parts other than the arm.
- Take care to choose the correct size. (Refer to the size chart on the package.)
- Do not use the product while sleeping. If the product moves into an unsuitable position or is used for long periods during sleep, it could interfere with blood circulation.
- Verify that this product will not cause any skin problems due to compression if it is used for a longer period of time.
- Take care not to wear this product inside out or upside down.
- Take care to put on this product in accordance with the steps described in "How to Apply".
- If the product moves out of position or becomes too loose, re-apply the product, beginning with the first step in "How to Apply".
- Do not wear the product folded. It could interfere with blood circulation.
- If the product is broken or worn out, discontinue use.
- Do not remanufacture, modify or repair the product. The product could lose its effectiveness and cause injuries.
- Wear the product directly against the skin and not over any other clothing.
- If you experience pain, swelling, skin irritation, discomfort or any other abnormal reaction while using this product, discontinue use and consult with a physician.

To prevent product damage:

- The product may become discolored by perspiration or other liquids. The product's dye may bleed onto clothing worn under or near the product.

How to Apply:

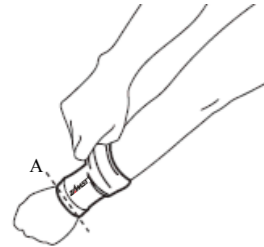
* This product can be used for the left or right arm. Illustrations are for the left arm.

1. Gather up the sleeve, put left arm into the sleeve and position the end of the sleeve even with the wrist (A) (Illustration 1).
*The "ZAMST" logo should fall along the upper side of the wrist.
2. Insert your right thumb into the sleeve and pull the sleeve upward evenly, taking care not to leave any creases or twists (Illustration 2).
3. Pull the sleeve upward into position over the mid-arm (Illustration 3).
*Do not pull the sleeve up under the armpit.
4. Verify again that there are no creases or twists. Try again if you experience discomfort.
*Creases or twists may form if you pull hard on a part of the product.

How to wash:

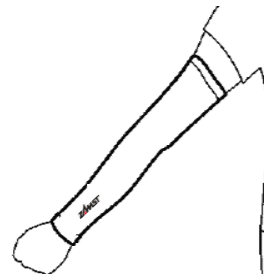
- Hand wash with mild detergent and air dry away from direct sunlight.
- Do not place in dryer. Do not dry clean or iron. Do not use products containing chlorine, including bleach. These may damage the product.
- The product's dye may bleed. Do not wash with whites or bright colors.

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